



Zono X用Sento
ユーザーマニュアル



Zono X用Sento ヘッドバンドセット

Zono X との組み合わせでマウスの左クリックやドラッグができる非接触型呼気スイッチです。



準備

Zono X用Sento ヘッドバンドセットは「ヘッドバンド」と「呼気センサーアーム付マグネットホルダー」に分かれています。

ヘッドバンドをホルダーにしっかり差し込んでください。2ページの写真のように正しい向きになっていることを確認してください。

Zono Xをホルダーに取り付けるタイミングは、Sentoを頭部に装着する前後どちらでも可です。Zono Xをホルダーに近づけると、磁石によって正しい位置に固定されます。取り外しも簡単です。

ホルダーに取り付けてからZono Xの電源を入れます。電源がオンになると、Zono XからSentoへ給電されて左クリックとして使用可能になります。




使用方法

ヘッドバンドが頭にフィットするよう幅を調整します。マグネットホルダーが頭部右側にき、平らな部分が正面を向くよう装着します。

呼気センサーはアームの先端にあり、ユーザーが吐く息（呼気）を検出します。センサーへ送る空気の量が多いほど入力（クリック実行）の精度が高まるので、呼気が当たりやすい位置にセンサーを配置してください。

口とセンサーの距離は1～5cm離すのが目安です。センサーの位置や角度によって呼気の当たりやすさが変わります。はじめは少し時間をかけて最適なポジションを見つけてください。

スイッチの動作テスト：センサーに息を吹きかけて左クリックが実行されるかどうか、長く息を吹きかけてドラッグが実行されるかどうか、を確認してください。

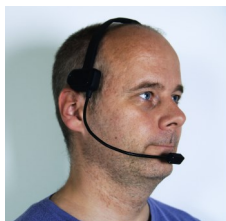

トラブルシューティング

息を吹きかけてもクリックが実行されない（されづらい）

吐く息が十分に呼気センサーに届いていない可能性があります。4ページを参照して、呼気センサーの適切な位置、角度を探してください。

意図せずクリックが実行されてしまう

話すときに吐く息をセンサーが拾ってしまっていると考えられます。これを避けるには呼気センサーをあごの前、下唇より少し低い位置に置くか、或いは呼気センサーを口から少し離すことをお勧めします。



Sentoを装着する際に、呼気センサーが衣服や皮膚と接触することで、感知してしまうこともあります。これを避けるには、Sentoを頭部に装着した後Zono Xをマグネットホルダーに固定してください。

クリーニング

Sentoは非接触型なので、お手入れも簡単です。呼気センサーに埃や唾液が入ってしまった場合は、電子機器に適した食品用洗浄スプレーを使って清掃してください。

ヘッドバンドのプラスチック部分は、湿らせた布と中性洗剤で拭いてください。

安全のための注意事項

このデバイスは磁石を内蔵しています。ペースメーカー、補聴器、クレジットカード、機械式時計、データ記憶媒体など、磁気に敏感な機器に近づけると破損する恐れがありますので、近づけないでください。



Product Disposal

When the device has reached its end-of-life it should be recycled as electric and electronic waste.



European Compliance

The manufacturer of this product confirms that this product complies with the European Union regulations for safety, health, environmental and consumer protection.



UKCA

UK RESPONSIBLE PERSON

AVENTIDO Limited, Bridgewater House, Century Park,
Caspian Road, Altrincham, Cheshire, WA14 5HH, United Kingdom



FCC Compliance

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.



保証期間

保証期間はご購入日から2年間です。保証書は箱の中に入っています。
保証書の全文はQuhaのホームページ (www.quha.com/warranty)でご確認ください。

© 2023 Quha oy. Quha oy is not responsible for possible errors in this user manual. Quha oy reserves the right to revise the content of this user manual.



Designed and manufactured in
Finland by Quha.

www.quha.com

Copyright 2023 Quha oy.

Quha, the Q-mark, Quha Zono X and Quha Sento are
trademarks or registered trademarks of Quha oy.